

**NOTE:** Reference the Cleaning Process section in the Product Manuals for further instruction. Cleaning and disinfecting an OAKWORKS product is a three part process.

1. Clean any visible soil off the surface.
2. Disinfect surface per manufacturer's recommended contact time.
3. Rinse off disinfectant with clean water and thoroughly dry with a clean cloth or towel.

## CLEANERS:

- Formula 409 (EPA No.: 67619-10)
- Fantastik (EPA No.: 89900-3)
- Windex Multisurface (EPA No.: 4622-593)
- Force of Nature (EPA No.: 93040-1)

## DISINFECTANTS:

- Formula 409 (EPA No.: 67619-10)
- Parker Labs Protex (EPA No.: 6836-152)
- Madacide-FD (EPA No.: 1130-15)
- Clorox EZ Kill Wipes (EPA No.: 5984-10)
- Clorox Disinfecting Wipes (EPA No.: 67619-9)
- Clorox Healthcare VersaSure Wipes (EPA No.: 67619-37)
- Clorox Healthcare Fuzion (EPA No.: 67619-30)
- PureGreen 24 (EPA No.: 72977-3)\*
- Rejuvenate Disinfectant Wipes (EPA No.: 74559-3)
- Force of Nature (EPA No.: 93040-1)
- Kurland Fog It IQ

**Composition:** Aqueous solution of polyhexamethylene biguanidine, didecyl dimethyl, ammonium chloride, alkylbenzyl dimethyl ammonium chloride, non-ionic surfactants, color and fragrance

## RECOMMENDED:

Oakworks recommends the use of disposable "paper roll" over the exam table/chair.

## NOT RECOMMENDED:

- Citrus based cleaners
- Acetone
- Hydrogen Peroxide based cleaners/disinfectants
- Acetic Acid
- Phenolics
- Bleach
- Unapproved quaternary ammonias ("quats")
- Alcohol-based cleaners or disinfectants

All cleaners and disinfectants have the ability to degrade and damage the upholstery. This information is not guaranteed, however, following the recommended cleaner and disinfectant list and cleaning process will provide the best care for your OAKWORKS products.

FAILURE TO FOLLOW RECOMMENDATIONS WILL VOID  
THE WARRANTY ON THE PRODUCT.

***If you have any questions, call Customer Service at 717.235.6807***